


▶ NEW



ArgoCap™

For easy and precise APC treatment

- Angling of the probe for better handling and tissue accessibility
- Constant probe distance due to secure fixation
- Enables APC with injection without changing instruments

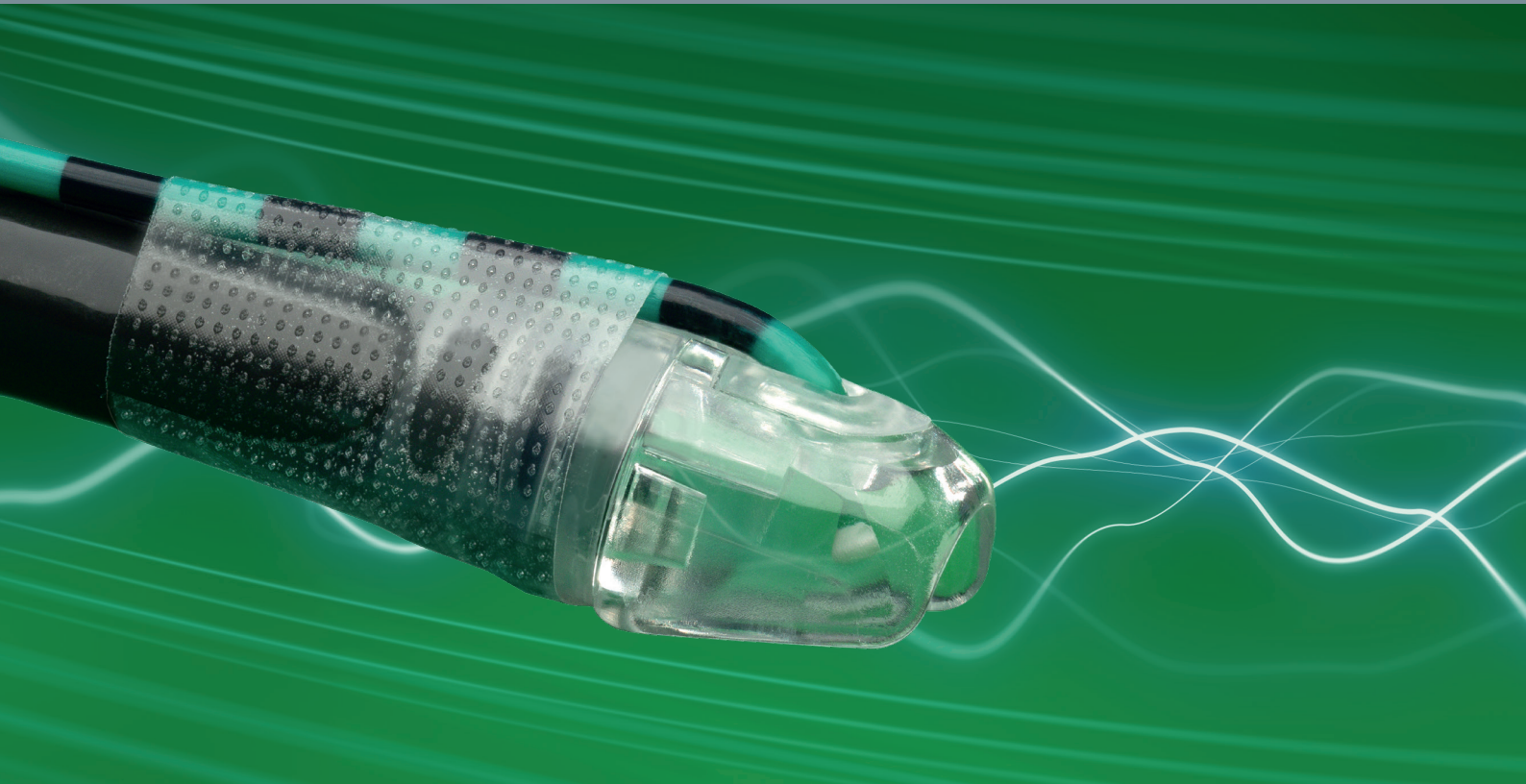



ArgoCap™ (Ref. no. 200.52)


The ArgoCap™ is an endoscopic cap for precise ablation during APC treatment of superficial lesions in the gastrointestinal tract.

The fixed positioning of the APC probe inside the cap enables secure guidance of the probe. The constant distance of the probe tip ensures homogeneous and focused¹ treatment during use and avoids over- or undertreatment at the same time.

The special cap design angles the probe, which makes it easier to handle during application and to reach the area to be treated. As the probe is guided along the outside of the endoscope shaft the working channel remains free, e.g. for simultaneous injection with an injection needle.

The ArgoCap™ is compatible with APC probes with an axial outlet and an outer diameter of 2.3 mm.



Dimensions

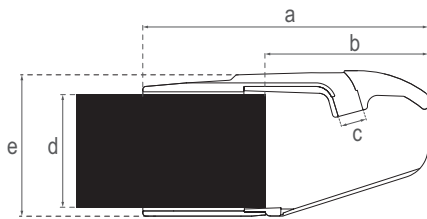



a Total length	28 mm
b Cap depth	16 mm
c Compatible probes Ø	2,3 mm
d Compatible endoscope diameter	9,2–10,5 mm
e Max. Outer diameter Ø	15,5 mm

Application


Barrett's oesophagus²

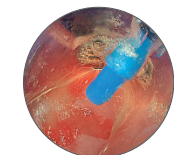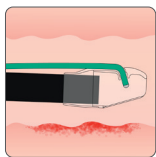



Inserting the endoscope with the mounted ArgoCap™

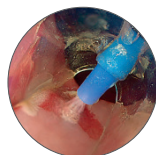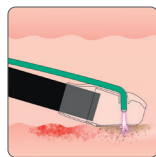

Localization of the area to be treated and start of APC treatment

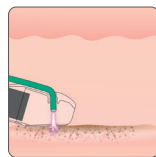

Further APC treatment of the remaining tissue areas

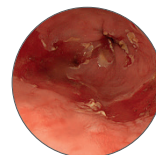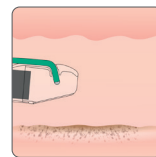

Successful APC treatment, removal of the endoscope

¹ Mueller J, Kayser G, Kuellmer A, Schiemer M, Bettinger D, Offensperger F, Thimme R, Schmidt A. **ArgoCap – feasibility and safety of a novel over-the-scope device to facilitate endoscopic APC treatment.** Minim Invasive Ther Allied Technol. 2023 Jun;32(3):103-111.

² Source: Prof. Dr. med. Benjamin Walter, University hospital Ulm, Germany

Ovesco Endoscopy AG

Friedrich-Miescher-Strasse 9
72076 Tuebingen
Germany

Phone +49 (0) 7071 96528-160

Fax +49 (0) 7071 96528-260

E-Mail service@ovesco.com

Ovesco Endoscopy USA Inc.

15300 Weston Parkway
Suite 101
Cary, NC 27513,
USA

Phone +1 919 651 9449

Fax +1 408 608 2077

customerservice@ovesco-usa.com

Scan here for more product information.

