




ovesco  
innovation in scope

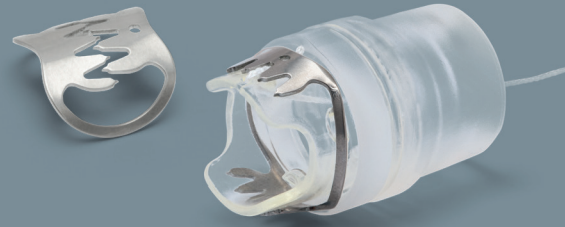

# stentfix OTSC® System.....

Preventing stent migration<sup>1</sup>

- Clip design adapts to the wall
- Allows proximal and distal fixation
- Dedicated cap shape for easy positioning






## ..... stentfix OTSC® System

The stentfix OTSC® System is the innovative solution for stent fixation in the digestive tract. It is a specially designed OTSC® System and labeled for the fixation of self-expandable metallic stents (SEMS).

The round design of the stentfix OTSC® clip adapts to the gastrointestinal anatomy. Due to the modified cap shape, stentfix OTSC® can easily be aligned to the stent opening for optimal stent mesh and tissue capture by the clip.

The compact design of the stentfix OTSC® System also allows the passage through a stent (from Ø 18 mm), therefore, fixation is possible either on the proximal or distal end.


### Measurements

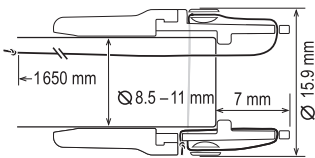


stentfix OTSC® System Set (ref. no. 100.50)

## ..... Application

Proximal fixation of the stent – Clinical example<sup>2</sup>

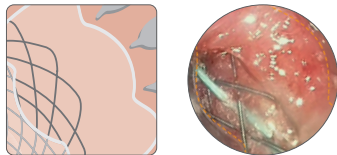


Align clip tooth rows parallel to stent opening, so that tissue and stent mesh are evenly captured. Mobilization of tissue by suction.

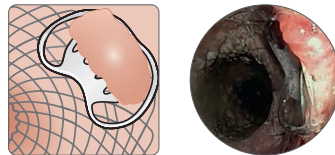


Optimal adaptation of the clip to the wall.

Distal fixation of the stent – Demonstration on Ex-vivo model

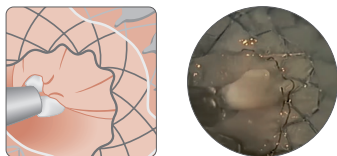

Align clip tooth rows parallel to stent opening, so that tissue and stent mesh are evenly captured. Mobilization of tissue by grasping below the stent meshes.

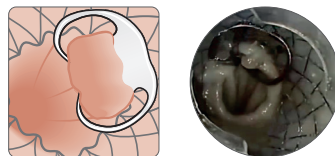

Optimal adaptation of the clip to the wall.

<sup>1</sup> Schiemer M, Bettinger D, Mueller J, Schultheiss M, Schwacha H, Hasselblatt P, Thimme R, Schmidt A, Kuellmer A. **Reduction of esophageal stent migration rate with a novel over-the-scope fixation device (with video).** Gastrointest Endosc. 2022 Jul;96(1):1-8.

<sup>2</sup> Dr. Massimo Conio, Sanremo Hospital, Italy

**Ovesco Endoscopy AG**  
Friedrich-Miescher-Strasse 9  
72076 Tuebingen  
Germany

Phone +49 (0) 7071 96528-160  
Fax +49 (0) 7071 96528-260  
E-Mail [service@ovesco.com](mailto:service@ovesco.com)

**Ovesco Endoscopy USA Inc.**  
15300 Weston Parkway  
Suite 101  
Cary, NC 27513,  
USA

Phone +1 919 651 9449  
Fax +1 408 608 2077  
[customerservice@ovesco-usa.com](mailto:customerservice@ovesco-usa.com)

Scan here for more product information.

